

SprintRay Ceramic Crown Polishing Guide

Prepare the crown after processing by reducing the supports and smoothing the surface. Crosscut carbide bur is recommended.*



Crosscut Carbide Bur

1. Pre-polishing with OptraGloss Spiral Wheel (8–10,000 RPM).



OptraGloss Spiral Wheel

2. Polish with Ivoclar Universal Polishing Paste using a goat hair brush (8–10,000 RPM).



Goat Hair Brush

3. Final buff with clean cotton buffing wheel on high speed (30–40,000 RPM).



Cotton Buffing Wheel

Result – smooth and glossy surface.



Materials Sources:

- OptraGloss Spiral Wheel (721094N) – Available from your Ivoclar distributor.
- Universal Polishing Paste (573660) – Sold directly from Ivoclar.
- Polishing Brushes and Buff Wheels – Available from a range of dental suppliers.